

## Clarity Vision Instructions

### Installation:

Note: Clarity Vision is calibrated to work with a 19 inch 1366/786 resolution monitor.

Plug the included power cable into the unit and then connect to the monitor with the included HDMI cable (if your monitor only has a VGA or D-Sub port, please use an HDMI to VGA connector which is not included).

Place two AAA batteries (included) into the remote and use the red power button on the remote to turn the device on.

After bootup, the home screen should appear. On the bottom there should a shortcut to the Clarity Vision application. This can also be found under the “Apps” section.

If you are having trouble with the remote responding intermittently, please ensure it is pointed in the direction of the device.

When running the app for first time, please access the “Settings” screen from the initial menu. Here you can select if a mirror is used and calibrate the device to feet from eye to monitor. If your room setup ever changes, please recalibrate the device.

### Remote Guide:



**Button “1”** - toggles letter optotype

**Button “2”** - toggles number optotype

**Button “3”** - toggles “tumbling E”

**Button “4”** - toggles “LEA Symbols”

**Button “5”** - toggles between single optotype or full line

**Button “8”** - red/green duochrome test

**Button “0”** - menu screen

**Left/Right** - change between 20/20, 20/25, 20/30, etc

**OK** - re-randomize optotypes