

Clarity Vision Instructions

Installation:

Note: Clarity Vision is calibrated to work with a 19 inch 1366/786 resolution monitor.

Plug the included power cable into the unit and then connect to the monitor with the included HDMI cable. (If your monitor only has a VGA or D-Sub port, please use an HDMI to VGA connector which is not included).

Place two AAA batteries into the remote and use the power button on the remote to turn the device on.

After bootup, on the home screen on the bottom there should be a shortcut to the Clarity Vision application. This can also be found under the “Apps” section.

When running app for first time, please access the “Settings” screen from the initial Menu. Here you can select if a mirror is used and calibrate the device to feet from eye to monitor. If your room setup ever changes, please recalibrate the device.

Remote Guide:



- Button “1”** - toggles letter optotype
- Button “2”** - toggles number optotype
- Button “3”** - toggles “tumbling E” optotype
- Button “5”** - toggles between single optotype or full line
- Button “0”** - menu screen
- Left/Right** - change between 20/20, 20/25, 20/30, etc
- OK** - re-randomize optotypes

